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There are mainly two types of therapeutic stockings on the market: circular knit and flat knit. The weaving of the circular weave with long extensibility is done with circular looms and the anatomical shape is given by varying the size of the stitches and the tension of the yarn, the number of needles from the thigh to the ankle does not change. The flat weave is elastic in both directions and has the stitching on the limbs because it is constructed in such a way as to obtain compressive supports with various shapes based on the lymphedema-lipedema found, thanks to the increase or decrease of the needles necessary for the number of stitches suitable for each specific anatomical part. The therapeutic function is to, through a mechanical action, move the edema and help lymphatic drainage to reduce its volume and consistency. The characteristics of the flat weave in the treatment of lymphedema are the shorter extensibility and the muscle pump effect given by the high ratio between working pressure and resting pressure. CzSalus garments respect these characteristics such as the variation in the number of needles and bi-elastic mesh and, moreover, with the addition of the embossed fabric we have improved the drainage of liquids with the pump effect resulting from the continuous micro massage, allowing to decrease, within two months of use, the centimetres of circumference of the calves, thighs and hips.

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